



Apply different waves and programs
in **2 muscular groups at the same time**

RUSSIAN + SQUARE + T.E.N.S. WAVES



NOVELTY
PROFESIONAL DEVICE

- BIPHASIC DEVICE
- 8 INDEPENDENT CHANNELS
- POWER CONTROL PER CHANEL
- DIGITAL DISPLAY
- START AND END ON RAMP
- 100% CONDUCTIVE ELECTRODES

THE MARKET'S NOVEL DEVICE

N 1108 Progressive® device is the most virtuous device for muscular stimulation applied in aesthetics, rehab, and spas manufactured by NOTO®.

It has the versatility of working with its 8 output channels at the same time or select 4 channels to work in a specific muscular group and the remaining 4 channels to work in another muscle group, at the same time with different programs and waves.

This feature allows professionals to work different zones of the patient at the same time, focusing on the type of therapy to be applied in that specific body area.

MICROELECTRONICS TECHNOLOGY

N 1108® Progressive output channels operates independently or combined, with an independent transformer per channel.

With 54 programs especially developed by professionals, N 1108 Progressive® grants professionals all possibilities to achieve visible, effective and durable results applying Russian, Square and T.E.N.S. Waves - all with current emission start and end on ramp.

Also, all parameters can be changed in a fast, safe way as well as creating and saving new programs permitting a patient follow-up, encouraging its progress and evolution.

NOTO® proposes an scenario were cutting-edge technology, quality products and supplies are fundamental for visible, effective, and durable results.

RUSSIAN WAVES

They are modulated medium frequency currents in low frequency signals. Its medium frequency has the objective of decreasing resistance or cutaneous impedance, allowing low frequency to act on the muscle.

The current objective is to aim for muscle toning or body contouring (modeling).

N 1108 Progressive® has the possibility to work this type of wave in bipolar or tetrapolar interferential way.

T.E.N.S.

T.E.N.S. utilize the nervous system mechanism to temporarily settle down acute and chronic pains.

The main advantage of applying T.E.N.S. is pain reduction, creating an immediate relief and without side effects.

SQUARE WAVES

They are presented in low and medium pulsed frequency. Square waves are the most energetic and powerful of stimulation currents.

MAIN INDICATIONS:

- Flaccidity and muscular contouring.
- Gluteus enhancement and hypertrophy
- Progressive muscular hypertrophy
- Disuse atrophy
- Post surgical strengthening
- Sports rehab
- Among others

MAIN INDICATIONS

- Peripheral nervous injuries
- Nervous comprehension injuries and reflex sympathetic dystrophy (carpal tunnel syndrome)
- Stump and/or "ghost members" pain.
- Neck and back pain
- Among others

MAIN INDICATIONS

- Functional training
- Sports
- Modeling
- Muscular strengthening
- Toning
- Lymphatic drainage
- Body Modeling
- Strength and resistance
- Among others


GADGETS INCLUDED WITH N 1108® PROGRESSIVE DEVICE:

				
8 Big Electrodes	8 Small Electrodes	12 elastic bands	8 Cables	1 Users manual

TECHNICAL SPECIFICATIONS OF N 1108 PROGRESSIVE:

MANUFACTURED BY:	NOTO GROUP S.A.
COUNTRY:	ARGENTINA
NAME OF THE PRODUCT:	N 1108 PROGRESSIVE
WIDE:	185 MM
LENGTH:	260 MM
HEIGHT:	130 MM-75 MM
WEIGHT:	2,1 KG
POWER SUPPLY VOLTAGE:	98 A 240 VAC 50/60HZ
POWER:	21 WATT
FUSE:	1 AMPER QUICK-CUT
PROGRAMMING:	MICRO CONTROLLER
WAVE FORMS:	RUSSIAN, T.E.N.S & SQUARE WAVES
PULSE WIDTH:	0 A 90 MA . IMPEDANCE 1K
CHANNELS QUANTITY:	8 INDEPENDENT FOR BODY APPLICATION
TYPE OF DEVICE:	IPX 01
FREQUENCY CARRIER	2500 HZ. RUSSIAN WAVES.

PROGRAMS:	34 PRESET
TIMER (TME):	1 TO 30 MINUTES
CONTRACTION:	2 TO 15 SECONDS
PAUSE:	2 TO 50 SECONDS
FREQUENCY:	2 TO 150 HZ
RELATIONSHIP:	1:1, 2:1 4:1
RISE RAMP:	1 SECOND
DECREASING RAMP:	1 SECOND
SEQUENCES:	1,2,3,4

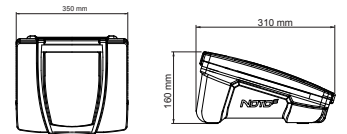

NOTE: DEVICE SPECIFICATIONS MAY VARY WITHOUT PREVIOUS NOTICE.

N 1108 PROGRESSIVE

NOTO supports all CUSTOMERS with a set of tools constantly available.

Tools as training, post-sale service support, and assured spare parts. All these instruments with the objective of making NOTO®'s products your best choice.

OTHER NOTO DEVICES:

DIOXI 1000 - CARBOXYTHERAPY WITH CONTROLLED TEMPERATURE SET-UP